



**PLEASE READ SECOND PAGE OF TEST REQUISITION FORM FOR POSSIBLE INTERFERENCES WITH CYREX TESTING.**

- 1 Before collection:**  
**DO NOT EAT OR DRINK ANYTHING** (with the exception of water) within 30 minutes of collection.  
**DO NOT BRUSH TEETH.** If brushed, wait one (1) hour before you continue with collection.  
**DO NOT APPLY** lip balms, ointments or lipstick.

- 2** Remove shrink band and wipe **EDGE** of the oral fluid tube using enclosed gauze pad.

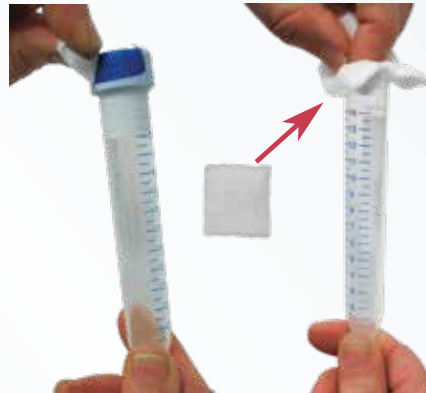

- 3** Naturally accumulate saliva in your mouth and spit into tube. Continue until 4 mL of **fluid** has been collected within a two (2) hour period of initial deposit into tube. Do not artificially stimulate saliva production, e.g. gum chewing.  
**Recap tube and gently invert a few times to mix specimen with preservative.**

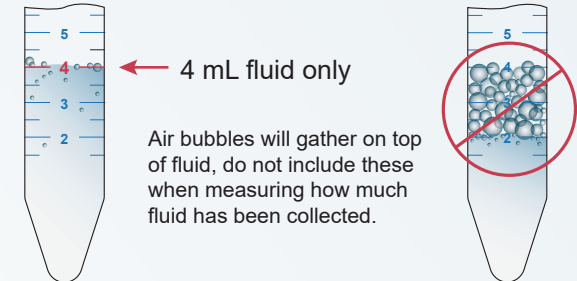

- 4** Wrap parafilm provided around the cap and the tube to prevent leakage during transit.




- 5** Every submitted specimen tube, per CLIA regulations §493.1240, must be labeled with:
- Patient's name **EXACTLY** as it appears on the Test Requisition form
  - Date of birth
  - Requisition number
  - Date and time of collection

Apply label onto oral fluid tube.



- 6** Write the date and time that you finished collecting your saliva specimen onto the front top right of your Test Requisition form.

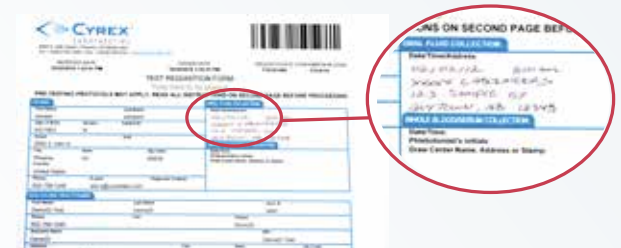

## SHIPPING INSTRUCTIONS

All specimens must be packaged properly for transit to the lab.

- 1 Place filled oral fluid tube into clear plastic "Biohazard Specimen Bag." Do not remove absorbent pad from bag.



Oral Fluid

- 2 Fold and seal "Biohazard Specimen Bag" and place it in the foam insert.



Oral Fluid



- 3 Place foam insert and Test Requisition form into the shipping box and close the box.



- 4 Insert the box into the white "UPS Laboratory Pak" and apply provided shipping label on the pack (if not already applied). Refrigerate specimen until ready to drop off.



- 5 SHIP ORAL FLUID (SALIVA) SPECIMEN THE SAME DAY COLLECTED FOR RECEIPT MONDAY THROUGH SATURDAY. SPECIMEN MUST BE RECEIVED WITHIN SEVEN DAYS OF COLLECTION. DO NOT FREEZE.